



# Rapid Test for Total Antibody to SARS-CoV-2 (COVID-19)

*Innovative  
High Performance  
Easy-to-use  
Rapid*




# An Innovative Rapid Test Designed by WANTAI

Wantai SARS-CoV-2 total antibody rapid test is intended for **screening of patients** suspected for infection with the SARS-CoV-2 virus, an aid in the diagnosis of the **COVID-19 disease**.

## Test Procedure



### 1. Prepare the Test

Allow the test cassette to reach room temperature, use it within 20 minutes after opening the pouch.

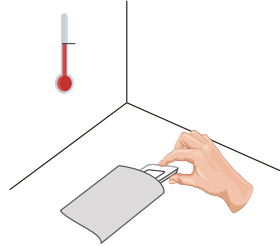


### 2. Add the Specimen

Add 10µl of whole blood, serum or plasma specimen.  
Add two drops of diluent buffer.

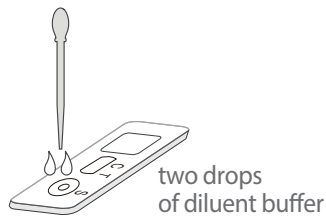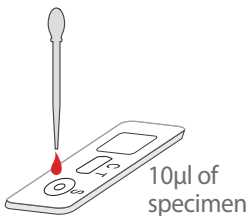

### 3. Read Result


Read the results at 15 minutes.

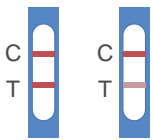

Positive



Negative

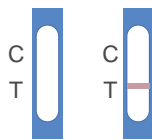

Invalid

\* Please refer to product insert for detailed instruction.

## Ordering Information

Components	WJ-2710	WJ-2750
Cassette	x 10	x 50
Diluent	x 1	x 5



**WANTAI BioPharm**

For prescription use only.

For in vitro diagnostic use only.

For use under Food and Drug Administration's  
Emergency Use Authorization (EUA) only.

## Advanced Diagnostic Technology

- Lateral Flow Test: suitable for rapid, on-site testing
- Easy-to-use: no special equipment. Works with whole blood, serum or plasma samples

### SARS-CoV-2 Total Ab Rapid Test Performance

Performance	Results	95% CI
Sensitivity	94.70%	89.38%-97.84%
Specificity	98.89%	96.80%-99.77%



**DYNEX Technologies, Inc.**  
Corporate Headquarters  
14340 Sullyfield Circle  
Chantilly, VA 20151-1621 USA

703.631.7800 Phone  
703.803.1441 Fax  
800.288.2354 U.S. Toll-free  
reagents@dynex.com



DYNEX® operates to ISO13485:2016 and 21 CFR 820.  
©2020 DYNEX. All rights reserved.